MicroDrop® Calimero2

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EN

Content

	Page
Product description	3
Complete equipment compressor	4
Complete equipment of the MicroDrop® Calimero2	5
Components of RF7 plus nebulizer	6
Warranty conditions	7
Important safeguards	8
Instructions for use	10
Assembly diagram	11
Using baby mask with angle	11
How to use the "RF7 Dual Speed Plus" nebulizer	12
Using the Duck-Clip	13
Cleaning, Sanitization and Disinfection	14
Compressor filter and changing of the compressor filter	17
Trouble Shooting	18
Disposal	19
Electromagnetic compatibility	19
Technical data	20
Symbols	21
Ordering Information	22

Product description

MicroDrop® Calimero2

Professional inhalation for children and adults

Congratulations on your purchase and thank you for choosing our unit.

Our goal is to fully satisfy customers by offering them cutting-edge systems for the treatment of respiratory tract ailments. Carefully read these instructions and keep them for future reference. Use the unit only as described in this instruction manual.

This is a medical device for home use and may only be used with medication prescribed or recommended by a doctor who has performed patient's evaluation.

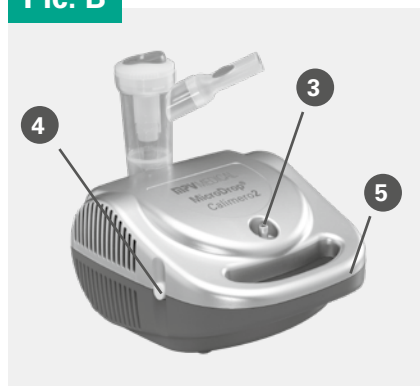
Visit our Internet site **www.mpvmedical.com** to view the whole range of **MPV MEDICAL** products.

Complete equipment compressor

Pic. A

Pic. B

Pic. C

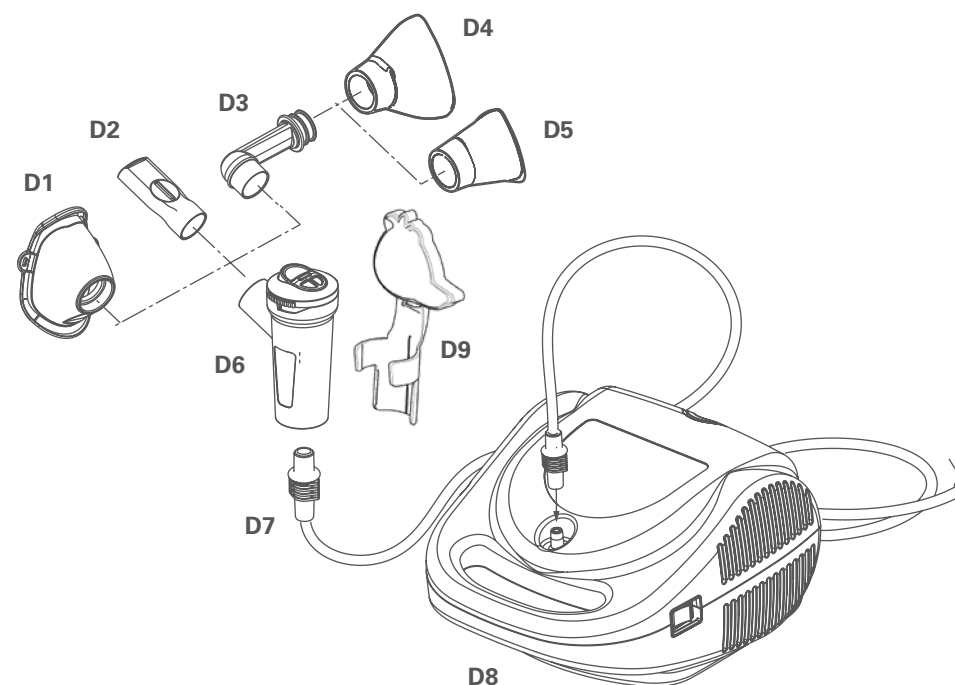
A1: Compressor
A2: On/off switch

B3: Air outlet port
B4: Air filter
B5: Carrying handle

C6: Nebulizer holder
C7: Cable compartment
C8: Power cord

Pic. D


Complete equipment of the MicroDrop® Calimero2

D1: Soft child mask (3 – 6 years)
D2: Mouthpiece
D3: Babyangle
D4: Siliconmask S (1 – 3 years)
D5: Siliconmask XS (0 – 12 month)

D6: RF7 plus nebulizer
D7: Tubing
D8: Compressor
D9: Duck-Clip

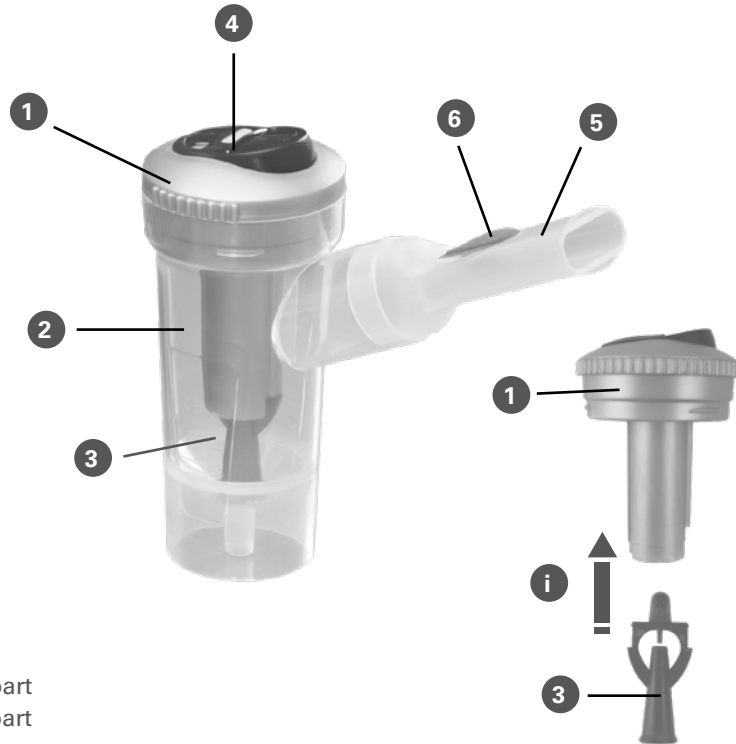Pic. E Components of RF7 plus nebulizer

- E1: Upper part
 E2: Lower part
 E3: Nozzle
 E4: Speed selector with valve system
 E5: Mouthpiece
 E6: Exhalation valve

Info (i):

Click nozzle (3) into the upper part (1)

Warranty conditions

- The device is covered by a **2-year warranty** as indicated on the packaging from date of purchase for any defects in the design or in the materials used.
- The warranty consists in the replacement and / or repair free of charge of originally defective components.
- The warranty does not cover the accessories supplied with the device and those parts subject to normal wear and tear.
- The device must be repaired by authorized technical service centres only.
- The device must be sent to the authorized service centre for repair within 8 days after the defect is noticed.
- Transport costs for the device are at user's charge.
- Any repair out of warranty conditions are at user's charge.
- The warranty does not cover failures resulting from misuse, abuse of the product or in case the damage is not due to manufacturer's liability (accidental fall, rough transportation, etc.).
- The warranty does not involve any compensation for anykind of damages to persons or things, either direct or indirect, occurred when the device is not operating.
- The warranty is valid from the date of purchase certified by the ticket or the invoice.

Important safeguards

Before using the device for the first time, and periodically during the product's life, check the integrity of its structure and of the power cable, making sure that there is no damage; if it is damaged, do not insert the plug and take the product immediately to MPV MEDICAL or to an authorized local dealer.

- This Medical Device is intended also for patient's direct use.
- The average lifespan of accessories is 1 year; however, we recommend you replace the nebulizer every 6 months in intensive use (or earlier, if the nebulizer is blocked) to always ensure maximum therapeutic effect.
- If the device is not performing according to the specification, please contact an authorized service centre or distributor.
- Children and disabled persons should always use the unit under strict supervision of an adult who has read this manual.
- Several parts of the device are small enough to be swallowed by children; therefore keep the device out of reach of children.
- Do not use the tubes and the cords of the device for purposes other than those specified as they may cause strangulation hazard; utmost attention must be paid to children and persons with special needs; usually, such persons are not capable of assessing the hazard correctly.
- The unit should not be used in the presence of anaesthetic mixture inflammable with air, oxygen or nitrous oxide.
- Keep the cord away from heated surfaces.
- Keep the power supply cord away from animals (for example rodents) as they may damage the insulation of the power supply cord.
- Do not handle the unit with wet hands. Do not use the device in humid environments (e.g. while taking a bath or shower). Do not immerse the device in water; should this accidentally occur, unplug the device immediately. Do not remove or touch the device immersed in water before pulling out the plug. Take the device immediately to an authorized service centre or to your local dealer.
- Only use the device in dust-free environments; otherwise, therapy may be compromised.
- Do not wash the device under running water or submerge in water. Keep away from splashing water or other liquids.
- Do not expose the device to particularly extreme temperatures.

- Do not put the device near heat sources, direct sunlight, or in hot environments.
- Do not obstruct or insert any objects in the filter or in its housing in the device.
- Do not obstruct the air vents placed on both sides of the unit.
- During use, always place the unit on a firm surface clear of obstacles.
- Check that there is no material obstructing the air vents before each use.
- Do not place any objects inside the air vents.
- Repairs, including the replacement of the power cord, must be performed by authorized personnel only following the information provided by the Manufacturer. Any unauthorized repairs will void the warranty and may pose a safety hazard for the user.
- The average duration provided for compressor is: 1000 hours.

Do not modify this equipment without authorization of the manufacturer.

- The Manufacturer, the Retailer and the Importer are responsible for the safety, reliability and performance of the device only if: a) the device is used in accordance with the operating instructions b) the electrical system of the premises where the device is used is in compliance with current laws.
- **Interactions:** The materials used that come into contact with medications have been tested with a wide range of medications. However, due to the variety and continuous evolution of medications, the possibility of a chemical interaction cannot be excluded. We recommend that once the medication has been opened you should use it as soon as possible and avoid prolonged contact of the medication with the nebulizer.
- The manufacturer should be contacted for reporting problems and/or unexpected events related to device operation and if necessary for clarifications on use, maintenance or cleaning.

MicroDrop® nebulizers are not suitable for use with essential oils.

Instructions for use

Before each use, the nebulizer and accessories should be cleaned according to the instructions in the section "Cleaning, sanitization and disinfection". This device is intended to aerosolise drugs and solutions suitable for inhalation treatment to be prescribed by the doctor. If the drug or solution is too dense, dilution with suitable saline solution might be needed, as prescribed by the doctor.

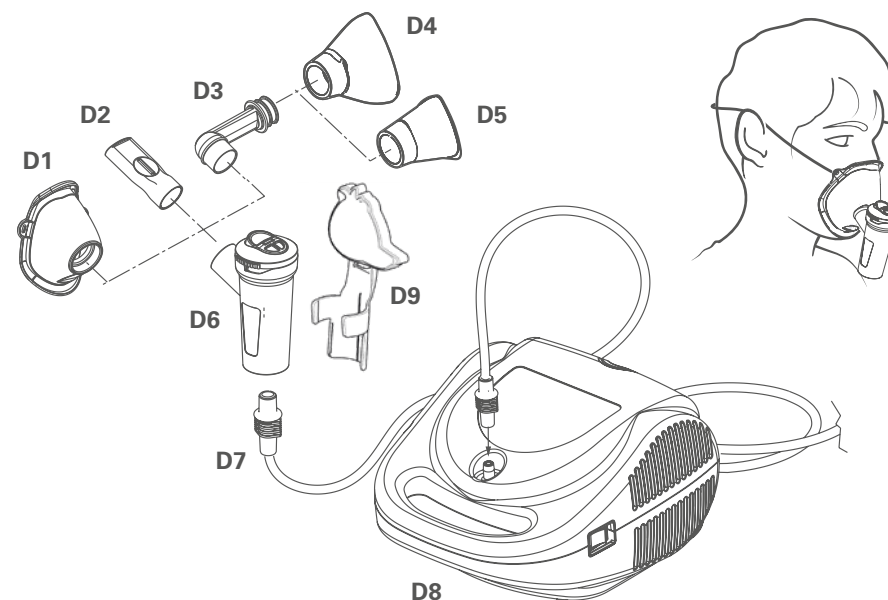
1. Insert the power cord (Pic. C8) into a power socket corresponding to the voltage of the device. This must be positioned so that it is not difficult to disconnect it from the mains.
2. Open the nebulizer by turning the upper part (Pic. E1) anticlockwise.
3. Pour the drug prescribed by the physician into the lower part (Pic. E2). Close the nebulizer by turning the upper part (Pic. E1) clockwise.
4. Connect the accessories as shown in the "Assembly diagram". Sit comfortably, holding the nebulizer in your hands, place the mouthpiece over your mouth or use the mask.
5. If you use the face mask, fit it to the face as shown in Pic. F.
6. Start the unit by means of switch (Pic. A2) and breathe in and out deeply through the mouthpiece or mask.
7. When the treatment is finished, turn off the unit and disconnect the plug from the mains socket.
8. It may happen that after a treatment session visible humidity deposits form in the tube (Pic. D7): disconnect the tube from the nebulizer and let it dry using the compressor air flow, as this prevents mould from growing in the tube.

A personal use of the nebulizer and its accessories is recommended in order to avoid any risks of infection.

After every use, store the unit and the accessories in a dry and dust free place.

Assembly diagram

Pic. F

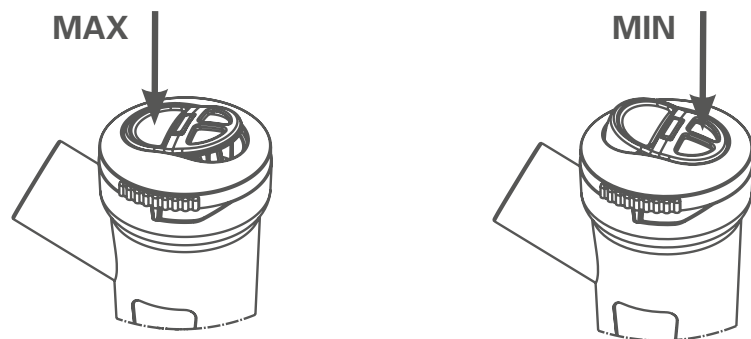
Using baby mask with angle

The MicroDrop® Calimero2 comes with different mask-sizes to give every child a perfect inhalation therapy. Please first check which of the enclosed masks fits your child best. The mask should cover the mouth and nose and lie tight on the face.

- Plug in the nebulizer with the baby angle and mask, as seen in the assembly diagram above.
- Bring the baby / toddler into a relaxed position
- Align the position of the angle and the position of the mask so that the nebulizer is always vertical. Otherwise medication could leak.
- For effective inhalation the mask should always be close to the face, so that the child can inhale sufficient aerosol. Therefore, apply the mask with light pressure over mouth and nose.
- If the baby initially defends against inhalation, it is helpful to directly take the mask in your hand (instead of the nebulizer), so you can follow the head movements better.

How to use the "RF7 Dual Speed Plus" nebulizer with speed selector and valve

The nebulizer is professionally fast and suitable for the administration of all types of drugs, including the most expensive ones, also for patients with chronic diseases. The geometry of the internal ducts of the RF7 Dual Speed Plus nebulizer ensures the ideal particle size providing effective treatment right down to the lower airways.



To speed up the inhalation therapy, please press on the MAX marking on the speed selector valve (Pic. E4).


To make inhalation therapy more effective press with your finger on the speed selector with valve system (Pic. E4) opposite to the MAX marking. In this way the valve systems in the nebulizer, in the mouthpiece and in the mask will ensure optimal medication delivery to the lower respiratory tract, limiting dispersion of medication into the environment.

SoftTouch masks (Pic. D1) has an outer edge made of soft biocompatible material that ensures excellent adherence to the face, and is also equipped with an innovative Dispersion Limiting Valve. These distinctive elements that distinguish it allow greater sedimentation of medication in the patient and also limit dispersion.

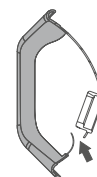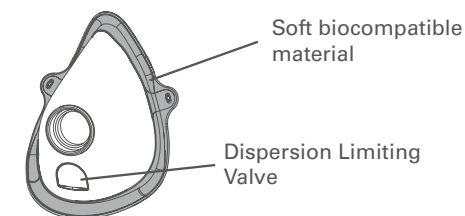SoftTouch Mask

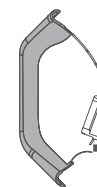During the inspiratory phase, the tab, acting as a Dispersion Limiting Valve, bends inwards towards the mask.


During the expiratory phase, the tab, acting as a Dispersion Limiting Valve, bends outwards from the mask.

Using the Duck clip

To make your child's inhalation therapy even more enjoyable, every Calimero2 comes with a duck clip (D9) called "Ducky".

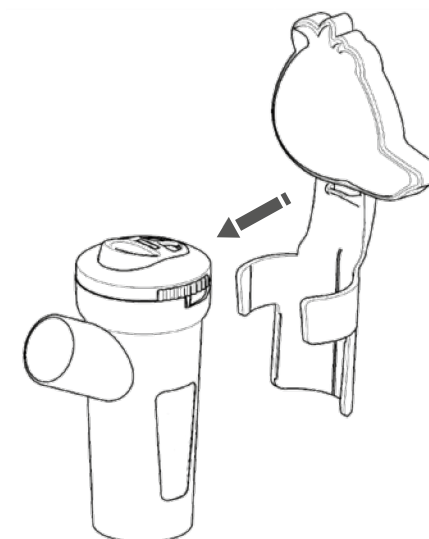He can be easily attached to the nebulizer, as seen in the picture. Why not telling your child the story of Ducky, who can blow magic clouds out of his scabbard, which will quickly cure children.

Cleaning, Sanitization and Disinfection

Clean the nebulizer and accessories as described below before and after each use. **Disinfect/Sanitize** at least once a day. Personal use of the nebulizer and its accessories is recommended to avoid the risk of infection.

1. Preparation

Open the nebulizer by turning the upper part (Pic. E1) counterclockwise. Pour away any remaining medication. Disconnect the nozzle from the top of the nebulizer (Pic. E1, E3). Remove the mouthpiece or mask or babyangle from the nebulizer. Rinse these parts under running water.

2. Cleaning

Outer surface of the compressor and tubing:

Use only a slightly damp cloth with an antibacterial detergent (non-abrasive and solvent-free).

3. Sanitization

Parts which can be sanitized: D1 (without rubber band) - D6 and E1 – E6

Method A (warm water or dishwasher)

Nebulizer and accessories:

Clean the accessories for about 5 minutes in warm water (about 40°C) and with mild (non-abrasive) detergent. Or put the parts in the dish rack of your dishwasher and start a 70°C hot rinse program. Do not use the dishwasher in combination with dirty dishes when cleaning the nebulizer.

Method B (vinegar solution)

Clean accessories by soaking in a solution of 50% water and 50% white vinegar for about 60 minutes. Then rinse thoroughly with warm tap water (about 40°C)

4. Disinfection

THERMAL DISINFECTION

Parts which can be disinfected: D1 (without rubber band) - D6 and E1 – E6

Method A (boiling water)

Disinfect the accessories by boiling in water for 20 minutes – preferably with distilled or demineralized water to prevent limescale. Please add enough water to the pot to prevent the nebulizer parts from melting by touching the bottom of the pot. After boiling, shake out the accessories and place them on a paper towel or kitchen towel, or dry with a warm jet of air (such as a hair-dryer).

Method B (baby bottles vaporiser)

Disinfect the accessories with a steam sterilizer for baby bottles (do not use microwave sterilizers). Perform the process exactly as directed by the sterilizer manufacturer's manual. To ensure that the disinfection is effective, choose a sterilizer with a cycle of at least 6 minutes.

CHEMICAL DISINFECTION

Disinfectable accessories are D1 (without rubber band) - D6 and E1 – E6

The disinfection procedure described in this section should be performed before using the accessory. Disinfection is only successful if every step is followed carefully and the components have been cleaned beforehand.

Note:

- a) Use a container of a suitable size to house all the components to be disinfected. Fill it with a 2% -5% sodium hypochlorite solution. Please strictly follow the instructions of the disinfectant manufacturer, especially regarding safety instructions, concentration, dosage and exposure time. Ready-to-use solutions that you do not need to mix yourself can be bought on the internet or in pharmacies.
- b) Immerse each component completely in the solution. Make sure that there are no air bubbles. These would prevent the disinfectant coming into contact with the components. The disinfection would not be completely possible.

- c) Remove the disinfected components after the exposure time and rinse thoroughly with warm drinking water respectively follow the instructions of the manufacturers instruction. After disinfecting, shake out the accessories and place them on a paper towel or kitchen towel, or dry them with a warm jet of air (such as a hairdryer). Dispose the disinfectant solution according to the manufacturer's instructions.

AUTOCLAVING / STERILIZATION (134 °C)

Accessories D1 (without rubber band) - D6 and E1 – E6 can be sterilized

Sterilization device: fractionated vacuum overpressure steam sterilizer according to EN 13060.

Procedure: Place each individual part to be treated in a sterile barrier system that meets the requirements of EN 11607. Place the packaged components in the steam sterilizer. Proceed the sterilization with a cycle of 10 minutes and a temperature of 134°C according to the operating instructions of the device.

Storage: Store the sterilized parts according to the instructions for use of the sterile barrier system.

The **sterilization procedure** has been validated in conformity with ISO 17665-1.

Use original manufacturer accessories and spare parts only.
The manufacturer will not be liable if non original accessories or spare parts are used.

Compressor filter and changing of the compressor filter

The compressor is equipped with an air inlet filter (Pic. B4), which filters out the aspirated air before it enters the unit. Dust and dirt particles do not get into the inhaled air from the nebulizer. Therefore, it is important to replace the filter as soon as it is dirty or takes on a different color.

Remove the filter from the compressor by simply pulling it outwards. Insert the new filter firmly into the opening provided in the compressor (Pic. B4).

Pic. B

Do not wash the filter and do not use it again. The regular inspection and replacement of the filter helps to ensure the perfect performance of the compressor. Do not remove the filter during operation of the device.

Trouble Shooting

Switch off the device before any procedure and unplug the power cable from the socket.

PROBLEM	CAUSE	SOLUTION
The device does not work	The power cable has not been correctly inserted into the power socket	Correctly insert the power cable in sockets.
The device does not nebulise or nebulises insufficiently.	The medication has not been inserted in the nebuliser.	Pour the right amount of medication in the nebuliser.
	The nebuliser has not been prop-erly fitted.	Disassemble the nebuliser and reassemble it correctly as shown in the assembly diagram on the cover.
	The nebuliser nozzle is clogged.	Disassemble the nebuliser, remove the nozzle and perform cleaning operations Failure to remove medication deposits from the nebuliser affects its efficiency and operation Strictly comply with the instructions contained in the "Cleaning, Sanitization and Disinfection" chapter.
	The air pipe is not correctly connected to the device	Ensure the device's accessories are properly connected to the air intake (see assembly diagram on the cover).
	The air pipe is bent, damaged or twisted	Unwind the pipe and ensure it is not crushed or punctured. If necessary replace it.
	The air filter is dirty	Replace the filter

Disposal

In conformity with Directive 2012/19/EC, the symbol shown on the appliance to be disposed of indicates that this is considered as waste and must therefore undergo "sorted waste collection". The user must therefore take (or have taken) the above waste to a pre-sorted waste collection centre set up by the local authorities, or else give it back to the dealer when purchasing a new appliance of the same type. Pre-sorted waste collection and the subsequent treatment, recovery and disposal operations favour the production of appliances made of recycled materials and limit the negative effects of any incorrect waste management on the environment and public health. The unlawful disposal of the product by the user could result in administrative fines as provided by the laws transposing Directive 2012/19/EC of the European member state in which the product is disposed of.

Electromagnetic compatibility

This device has been designed to satisfy requirements currently required for electromagnetic compatibility (EN 60 601-1-2:2015). Electrical medical devices require special care. During installation and use with respect to EMC requirements, it is therefore required that they be installed and/or used according to the manufacturer's specification. Potential risk of electromagnetic interference with other devices, in particular with other devices for diagnosis and treatment.

Radio and mobile telecommunications devices or portable RF (mobile phones or wireless connections) may interfere with the operation of electrical medical devices. The manufacturer reserves the right to make technical and functional changes to the product without notice. For additional information please visit www.mpvmedical.com.

Technical data MicroDrop® Calimero2 (Mod. P1207EM F1000)

Voltage	230V~ 50Hz 140VA
In conformity with	93/42/EWG, CE0051
Max Pressure	2.6 ± 0.4 bar
Compressor air output	10 LPM approx.
Noise level (at 1 m)	54 dB (A) approx.
Operation	Continuous use
Dimensions (W)x(D)x(H)	170 mm x 200 mm x 105 mm
Weight	1.2 kg

Applied parts

Type BF applied parts include:	(D1 –D6)
D1 - Soft child mask	PP+TPE
D2 - Mouthpiece with valve	PP + silicone
D6 - RF7 plus nebulizer	PP + silicone
D7 - Connection tube	PVC phthalates free
D4, D5 - Silicone masks	100 % silicone

RF7 plus nebulizer

Medication minimum fill volume	2 ml
Medication maximum fill volume	8 ml
Operating pressure (with neb.)	1.00 bar approx.
Delivery rate (min)	0.23 ml/min
Delivery rate (max)	0.53 ml/min

In vitro testing performed by TÜV Rheinland Italia S.r.l. in collaboration with the University of Parma and in compliance with the EN 13544-1: 2007 + A1 European Standard for aerosol therapy devices Standard. Values gathered with mouthpiece. More details are available upon request.

MMAD	MIN 2.76 µm / MAX 2.91 µm
Fine Particle Fraction < 5 µm	MIN 78.5 % / MAX 76.0 %

Operating conditions

Temperature	min. 10° C; max. 40° C
RH Air humidity	min. 10 %; max. 95 %

Storage conditions

Temperature	min. -25° C; max. 70° C
RH Air humidity	min. 10 %; max. 95 %
Operating/storage atmospheric pressure	min. 69 KPa; max. 106 KPa

data measured according to Flaem 129-P075 internal procedure

Symbols

Class II unit

Type-BF applied parts

Important: check the operating instructions

Alternate current

TÜV Certification

Switch OFF

Switch ON

EC Marking medical ref. Dir 93/42 EEC and subsequent updates.

In compliance with: European Standard EN 10993-1 „Biological Evaluation of medical devices“ and European Directive 93/42/EEC „Medical Devices.“ Phthalate-free. In compliance with: Reg. (EC) no. 1907/2006

IP21

Enclosure protection rating: IP21.
(Protected against solid bodies over 12 mm.
Protected against access with a finger; Protected against vertically falling water drops.)

Minimum / Maximum Temperature

Minimum / Maximum Humidity

Minimum / Maximum Atmospheric pressure

Ordering Information

MicroDrop® Calimero2

Professionell nebulizer for babies and children

REIMBURSEMENT-NO. 14.24.01.0129

Including: Compressor, RF7 plus children's nebulizer, mouthpiece with exhalation valve, tubing, baby angle, silicone mask size XS (Babies 0 – 12 months), silicone mask S (children 1 – 3 years), soft mask child yellow, duck clip, spare compressor filter, practical carrying bag and instructions for use

PRODUCT	PZN**	REF-NO.
MicroDrop® Calimero2	08628235	M51504-00
Manual	---	M50010-55800

MicroDrop® Profi AV-Set child universal

Including: RF7 plus child nebulizer, mouthpiece with exhalation valve, tubing, universal adapter (for connection to other compressors*), Soft-kids-mask, filter compressor, manual

PZN	REF-NO.
00347459	M53700-12

MicroDrop® Calimero2 Year-Set child universal


Including: RF7 plus child nebulizer, mouthpiece with exhalation valve, tubing, universal adapter (for connection to other compressors*), compressor filter and instructions for use

PZN	REF-NO.
08448869	M53700-17

MicroDrop® RF7 plus children nebulizer with mouthpiece and exhalation valve

PZN	REF-NO.
08628241	M53500-03

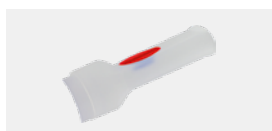
MicroDrop® mouthpiece RF7 plus children nebulizer with exhalation valve

PZN	REF-NO.
10002164	M50020-52500

MicroDrop® RF7 plus softmask child yellow / transparent

PZN	REF-NO.
00348476	M50020-44200

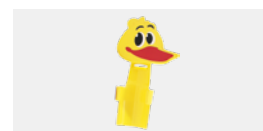MicroDrop® Duck-Clip duck-face click on for RF7 plus children nebulizer

PZN	REF-NO.
10124714	M53500-05

MicroDrop® tubing 2m with 2 fits-all connectors phthalate and BPA free

PZN	REF-NO.
09937470	M50020-42400

MicroDrop® RF babyangle / adapter with exhalation hole for usage of silicone masks

PZN	REF-NO.
00347703	M50020-40500

MicroDrop® silicone mask size XS/S 100% silicone ID 22 mm with exhalation valve

	PZN	REF-NO.
Size XS (0 – 1 years)	01002834	M50020-43500
Size S (1 – 3 years)	01002828	M50020-45900

MicroDrop® 2 filter compressor 1 pc. for compressor

PZN	REF-NO.
09937530	M50010-42800

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Hersteller / MANUFACTURER (93 / 42 / EEC):
Designed und entwickelt von:
DESIGNED AND ENGINEERED BY:

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FLAEM NUOVA S.p.A., Via Colli Storici,